

Bioline™ SYPHILIS 3.0


SYPHILIS ANTIBODY TEST

Bioline™ Syphilis 3.0 test is a solid phase immunochromatographic assay for the qualitative detection of antibodies of all isotypes (IgG, IgM, IgA) against *Treponema pallidum* (TP) in human serum, plasma or whole blood.

- Qualitative immunochromatographic assay
- No need for preprocessing and equipment
- Shelf life and storage temperature: 24 months from the date of manufacturing at 2-30 °C
- Performance: Sensitivity 99.3 %, Specificity 99.5 % (vs TPHA)

MATERIALS PROVIDED


- Test device / Multi-device / Strip
- Assay diluent
- Option: Lancet, alcohol swab, capillary pipette

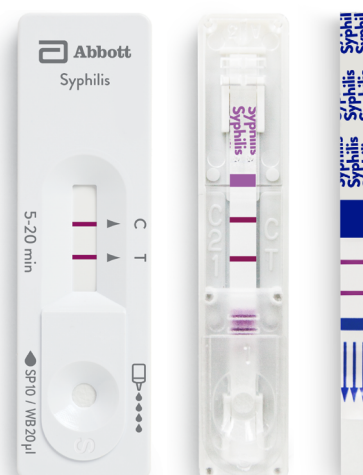

SIMPLE PROCEDURE

1 Add Specimen


Dispense 10 µl of plasma, serum or 20 µl of whole blood into the specimen well.

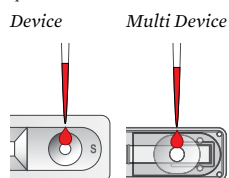

2

Add Assay Diluent


Dispense 4 drops of the assay diluent.

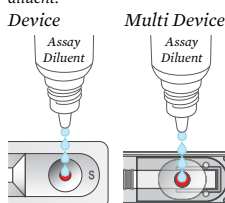

Wait 5-20 mins.

1 Add Assay Diluent and Specimen

Add 10 µl of serum or plasma or 20 µl of whole blood and 4 drops of assay diluent to the empty test tube. And then, mix well.

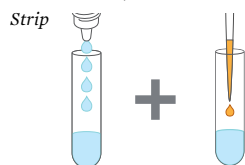


2

Insert Strip

Insert strip into the test tube.

Strip

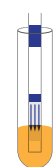

RESULTS INTERPRETATION

NEGATIVE

POSITIVE

INVALID

The presence of any test line, no matter how faint, the result is considered positive.

ORDERING INFORMATION

PRODUCT	CAT. NO.	TYPE	SPECIMEN	PACK SIZE
Syphilis 3.0	06FK10	Device	Serum/Plasma/Whole blood	30T/Kit
Syphilis 3.0	06FK11	Multi-Device	Serum/Plasma/Whole blood	10Tx10/Kit
Syphilis Fast 3.0	06FK12	Strip	Serum/Plasma/Whole blood	25T/Kit
Syphilis 3.0*	06FK13	Device	Serum/Plasma/Whole blood	25T/Kit
Syphilis 3.0**	06FK16	Device	Serum/Plasma/Whole blood	25T/Kit
Syphilis 3.0***	06FK17	Device	Serum/Plasma/Whole blood	25T/Kit

(*) Lancet, Capillary pipette included.

(**) Lancet, Capillary pipette, Alcohol swab included.

(***) Safety lancet, Capillary pipette, Alcohol swab included.